

# ***Slipp***®

## Simple Instructions for using SLIPP® - The Patient Mover

**INSTRUCTIONS FOR CLEANING:** Clean SLIPP® with any anti-microbial or germicidal solution or with Bleach Solution 5% or 10%. *Be sure you dilute any cleaner to manufacturer's instructions.* Avoid using any solution with iodine as it will stain the SLIPP® material. If the SLIPP® becomes extremely soiled, it can be machine washed however it *cannot* be machine dried.

**SLIPP CARE:** The Slipp® should be inspected on a regular basis. Each Slipp® is sealed with fluid and air inside of the unit. When inspecting the Slipp®, if no air is present inside the Slipp® it has been punctured and needs to be properly discarded. The fluid inside the Slipp® is extremely slick. In case of puncture, any spilled lubricant and surface area the fluid comes in contact with should be cleaned immediately with isopropyl alcohol and the unit needs to be properly discarded. If the lubricant inside the Slipp® comes in contact with skin or clothing wash with soap and water.


**WARRANTY:** The manufacturer offers a one year warranty against defects in material and workmanship.

**CUSTOMER SERVICE:** The instructional DVD for Slipp® the patient mover can be viewed at [www.wrightproductsinc.com/video](http://www.wrightproductsinc.com/video). It is our goal to have fully satisfied customers. If you have any questions regarding your SLIPP® or its use, please call our Customer Service Department at **1-800-356-6911**


**NOTE:** The force required to move a patient using SLIPP® is minimal. Do not pull hard, and do not lift. Remember to pull on just the draw sheet, *not the SLIPP®!* SLIPP® should be removed from under the patient after transfer or repositioning.

## FOR TRANSFERRING A PATIENT FROM BED TO GURNEY & VICE VERSA


1. Begin by lowering the side rails on the bed and gurney.

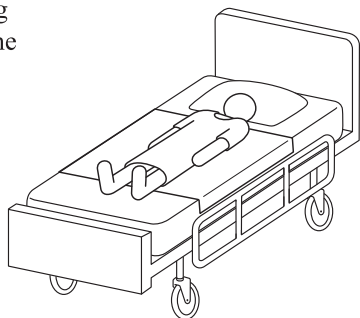


2. With a draw sheet under the patient, use the draw sheet to roll the patient onto his/her side opposite the desired direction of transfer.

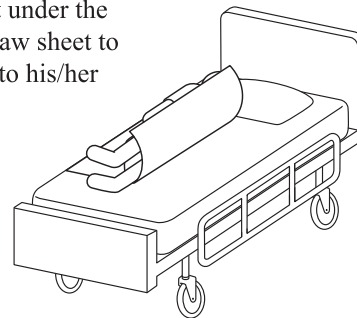


3. With the patient on his/her side, place the SLIPP® directly next to the patient so that it will be under the draw sheet and between the patient and the bed.

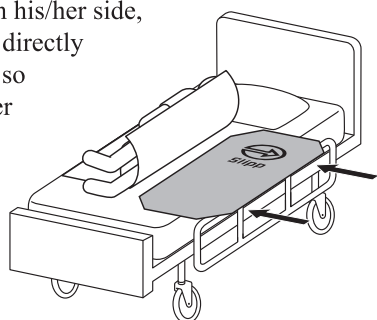


4. Roll patient back into original position onto his/her back and on top of draw sheet and SLIPP®.

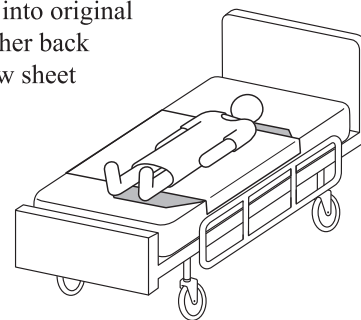


5. Position gurney next to bed. Both the bed and gurney should be locked in place for safety.

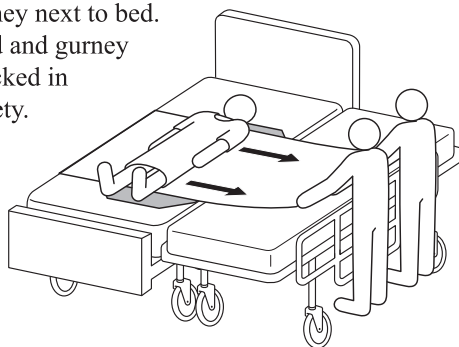


6. To complete the transfer, gently pull the draw sheet and the patient will glide easily across the bed to the desired destination.

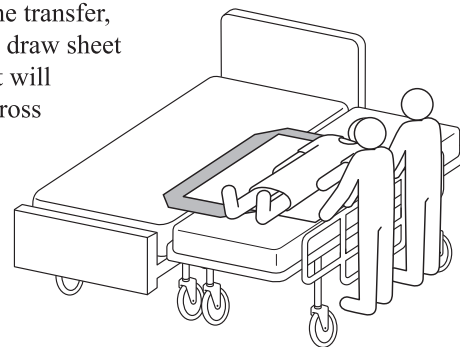

7. Raise the outside side rail on the gurney and unlock and separate bed and gurney. To remove the SLIPP®, use the draw sheet to roll the patient onto his/her side and remove the SLIPP®.

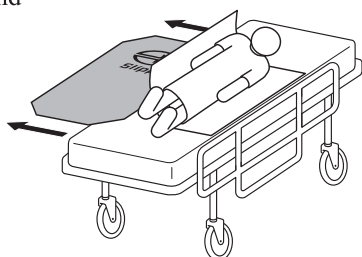


8. Raise the remaining side rail on the gurney and the patient is ready for transport.

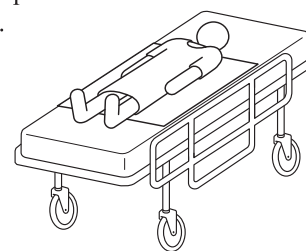


**NOTE:** The force required to move a patient using SLIPP® is minimal. Do not pull hard, and do not lift. Remember to pull on just the draw sheet, **not the SLIPP®!** SLIPP® should be removed from under the patient after transfer or repositioning.

## FOR REPOSITIONING OR MOVING PATIENT UP IN BED


1. Begin by lowering the side rails on the bed.

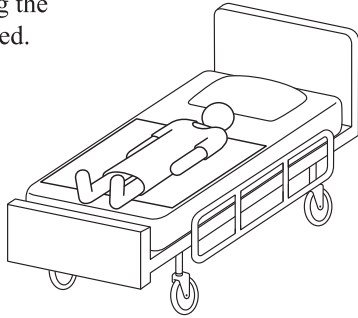


2. With a draw sheet under the patient, use the draw sheet to roll the patient onto his/her side.

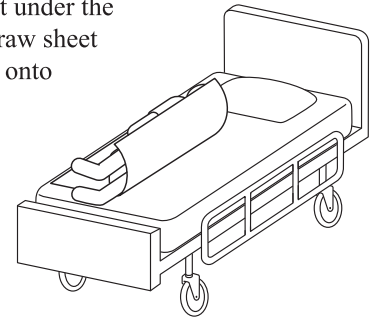


3. With the patient on his/her side, place the SLIPP® directly next to the patient and make sure that the top of the SLIPP® is above the shoulders and the bottom end is below the hips, i.e., so that both shoulders and hips are on the SLIPP®.

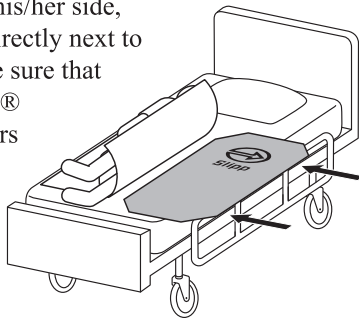


4. Roll patient back into original position onto his/her back and on top of draw sheet and SLIPP®.

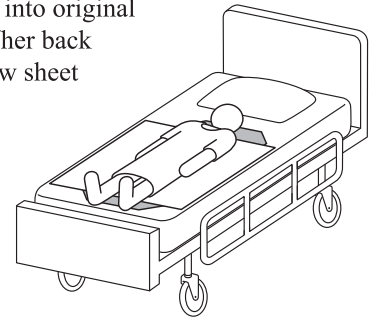


5. With the patient now lying on top of the draw sheet and the SLIPP®, utilizing two(2) nursing assistants, one on each side of the bed, you are now ready to reposition the patient.

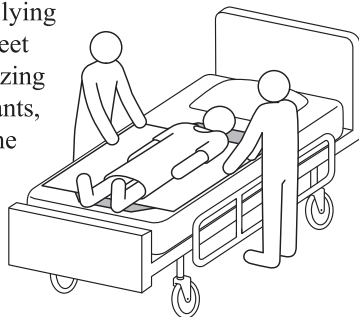


6. Grasp the draw sheet with both hands and **slowly** pull the patient up in the bed or toward the center of the bed as desired. **Do not pull hard, and do not lift.**

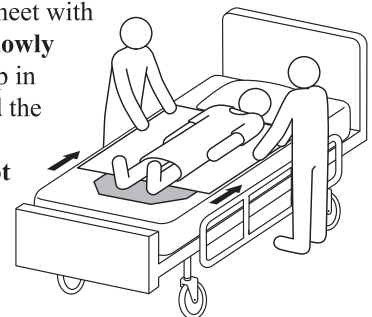

7. To remove the SLIPP®, use the draw sheet to roll the patient onto his/her side and remove the SLIPP®.

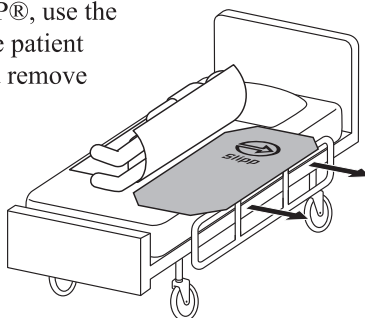


8. Gently roll patient back into original position on his/her back.

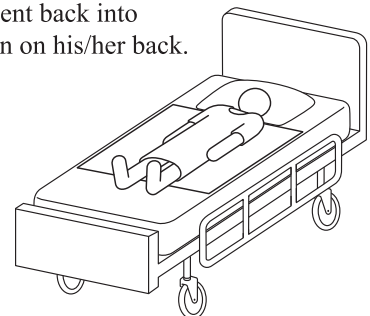


# FOR MOVING PATIENT FROM BED TO WHEELCHAIR

NOTE: Gait belt optional for the following procedure.


1. Using a draw sheet, begin by rolling the patient onto his/her side.

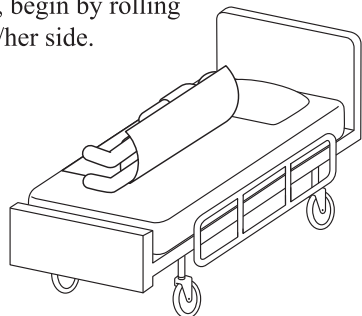


2. Place the SLIPP® directly next to patient. Make certain both the shoulders and hips of patient are on the SLIPP®.

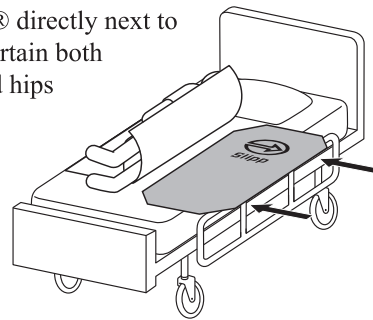


3. Roll patient back into original position onto his/her back and on top of draw sheet and SLIPP®.

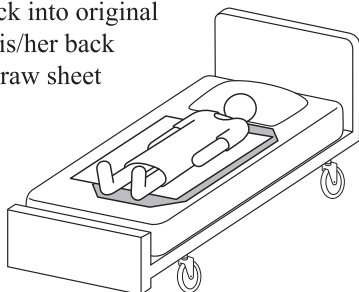


4. Set the patient up in bed making sure to firmly support him/her as you lean them forward.

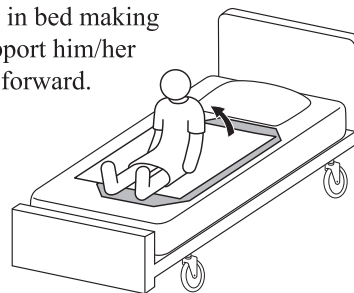


5. Pick patient's legs up and pivot the patient in the direction **opposite** the move.

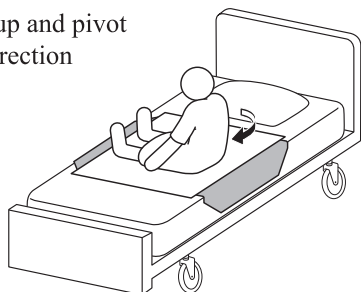


6. Place wheelchair next to bed **facing the bed** with wheels locked for safety.

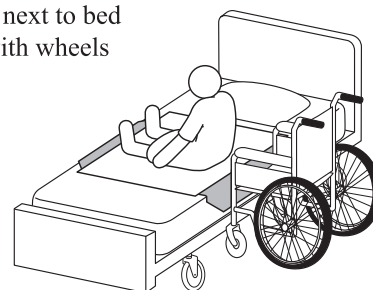

7. With one person on each side of the patient, using the gait belt or the draw sheet, gently slide patient into the chair.

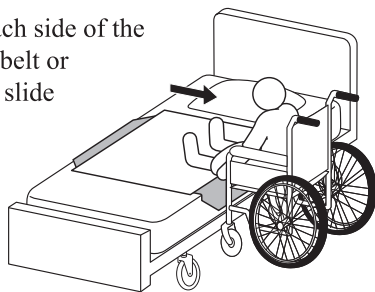

8. Unlock wheels and back slowly away from bed, and place patient's feet in wheelchair footrest.

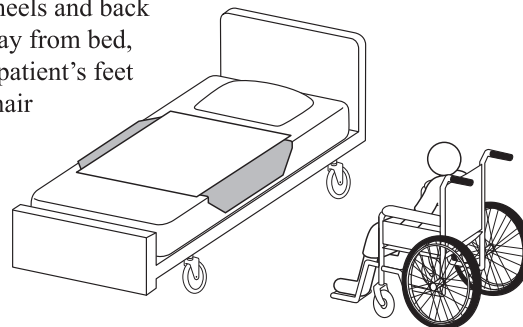

**SIMPLY REVERSE THIS FOR WHEELCHAIR TO BED TRANSFER**

FOR QUESTIONS OR TO ORDER CALL:

**1-800-356-6911**

Wright Products, Inc. 1909 S. Taylorville Road Decatur IL 62521

**e-mail: [gwslipp@aol.com](mailto:gwslipp@aol.com) [www.wrightproductsinc.com](http://www.wrightproductsinc.com)**

SLIPP® is a Registered Trademark of Wright Products, Inc., Decatur, IL 62521, U.S. & Foreign Patents

© 2017 Wright Products, Inc.